





Transperineal Biopsy Guide AMD SureFire Universal for Multiple Biplane

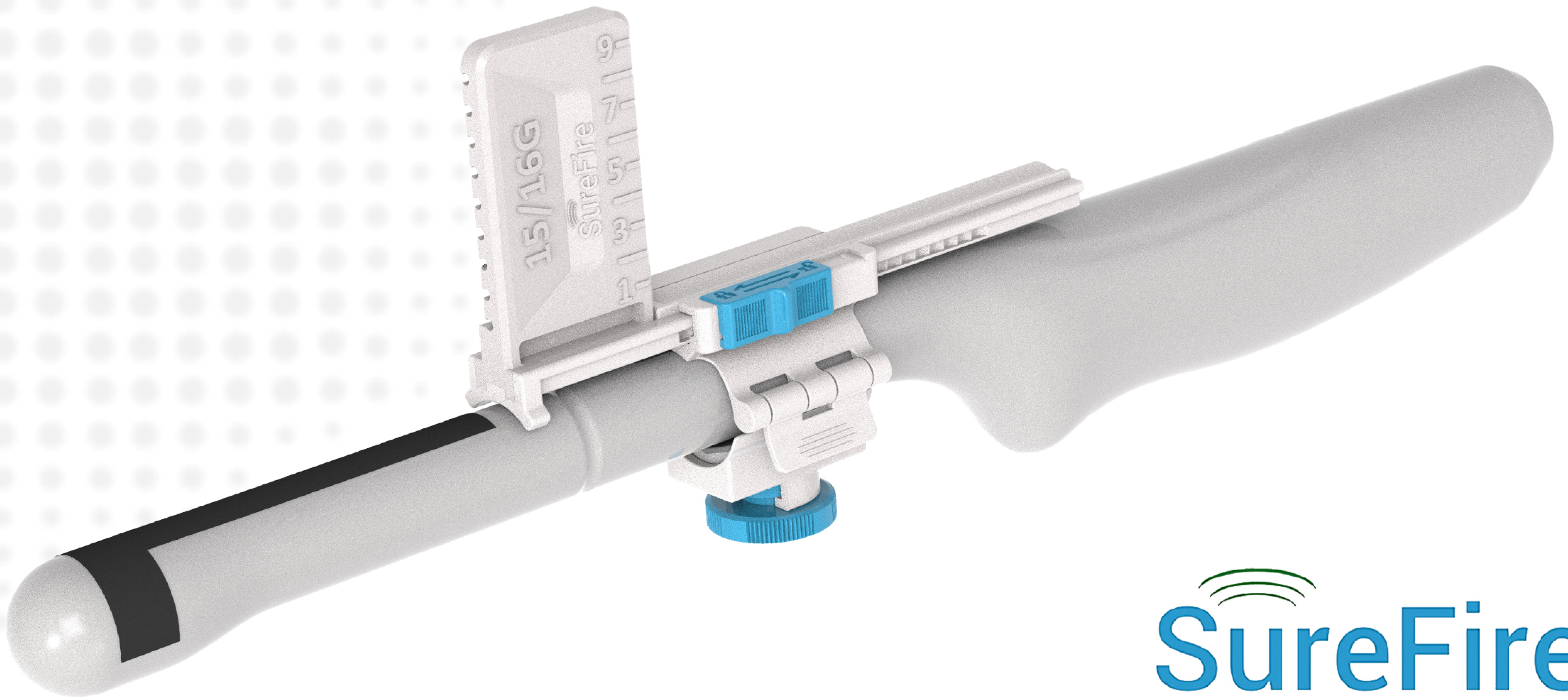

SureFire

Why SureFire Universal?


- Broaden procedures offered with your existing biplane probe
- Eliminate time consuming steppers

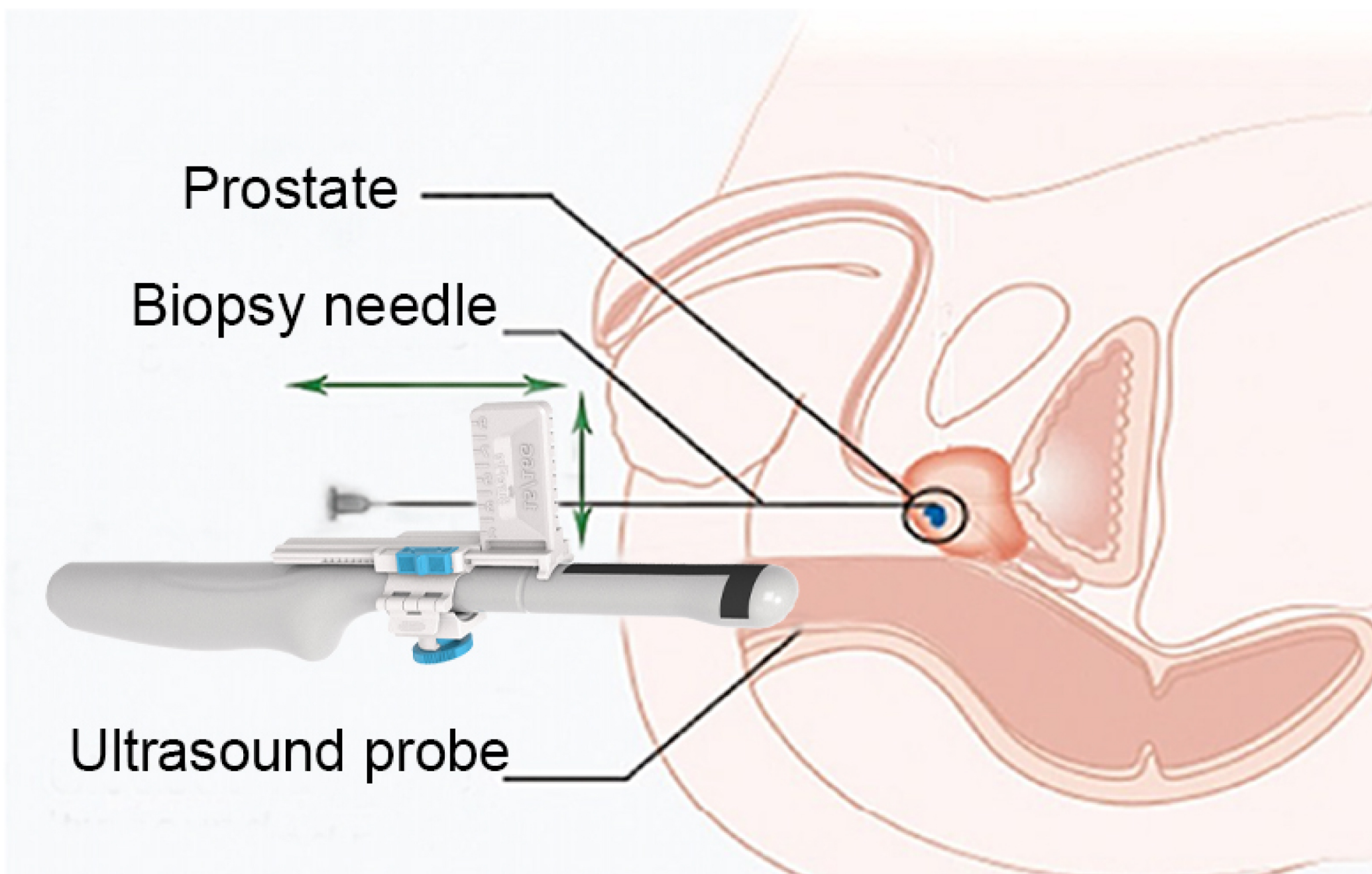

- Vertical access to 8cm. with 9 puncture channels

Compatible with:


- Canon
- GE
- Mindray
- Philips
- SonoScape
- Terason
- Siemens

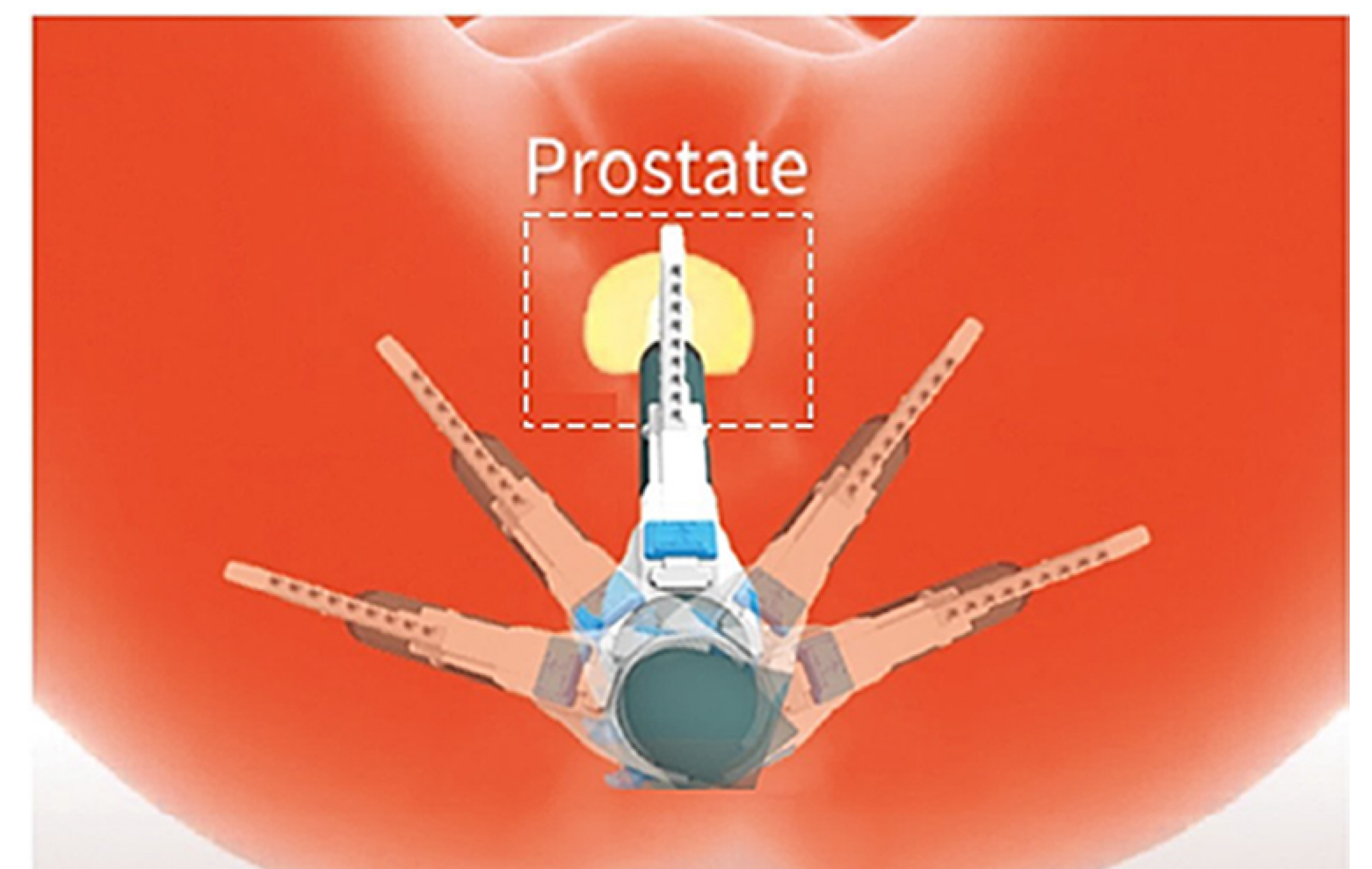



- Free Sectorial plane adjustment

Order information


Order No.	Description	Needle Size	Hole	Height	Packaging
28-TPU166	SureFire Universal for multiple Biplane Probes	15G-16G	9 Closed Channel	8cm	10ea/Case
28-TPU168	SureFire Universal for multiple Biplane Probes	17G-18G	9 Closed Channel	8cm	10ea/Case

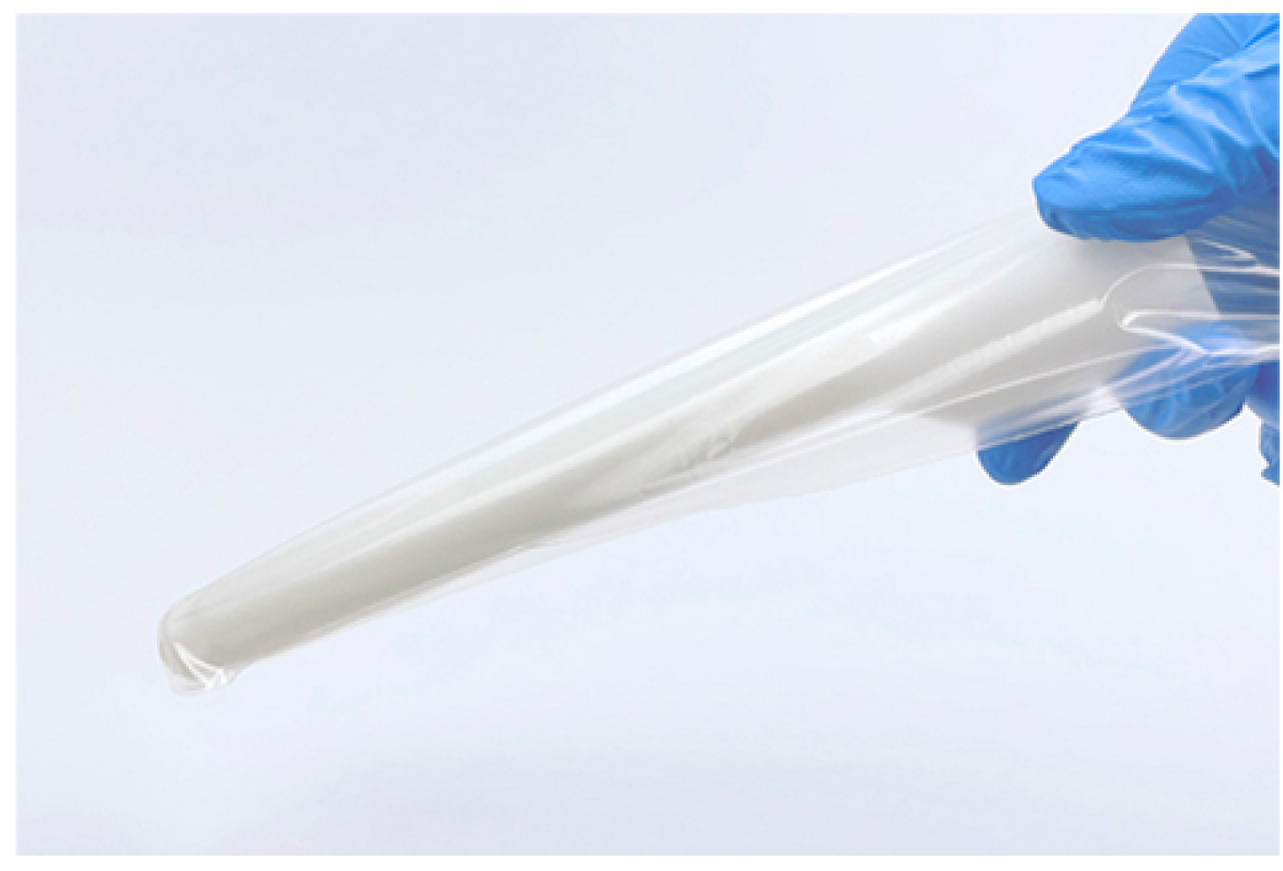


1. Apply gel on probe tip and place cover

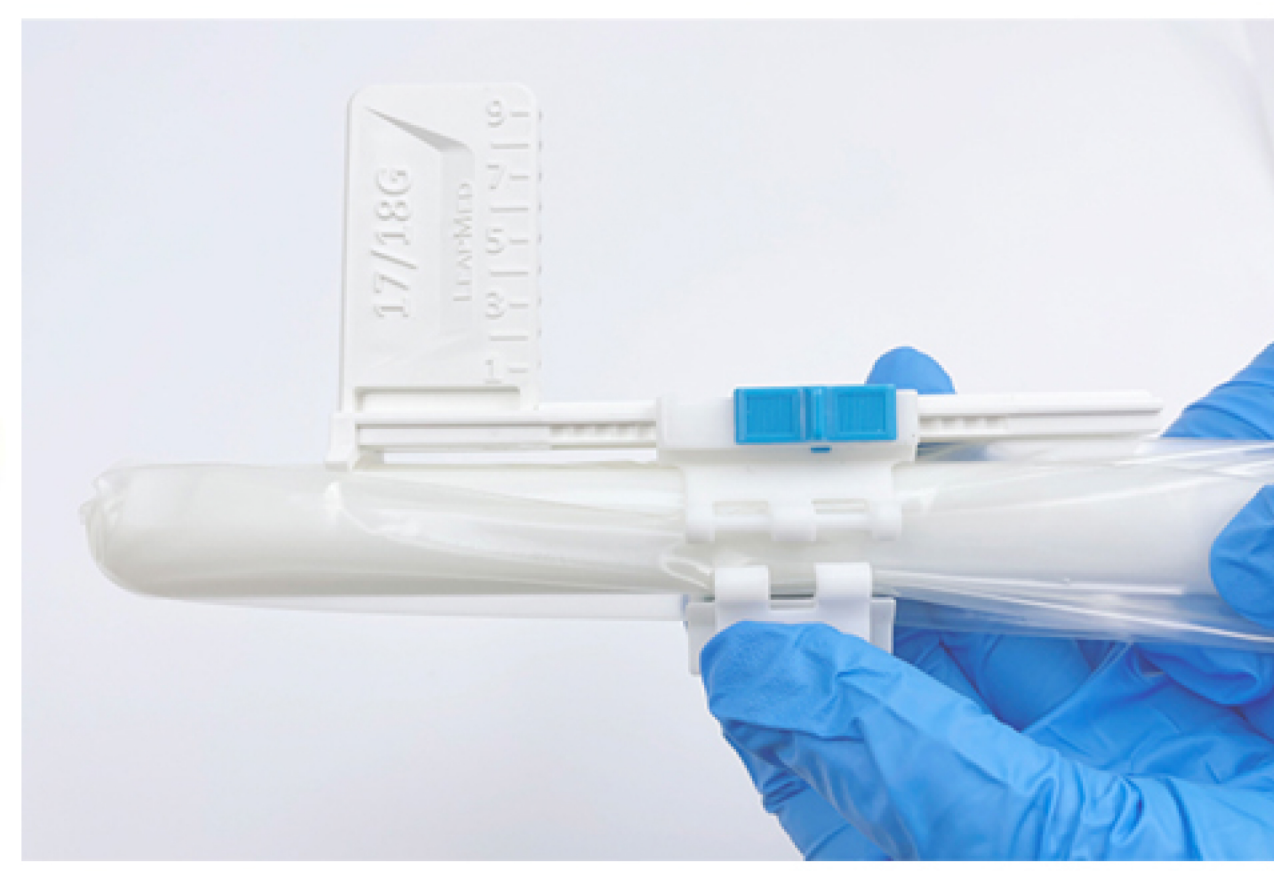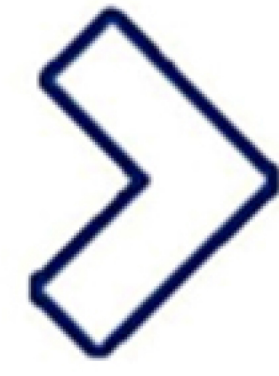



2. Press the left side of the locking arm to unlock

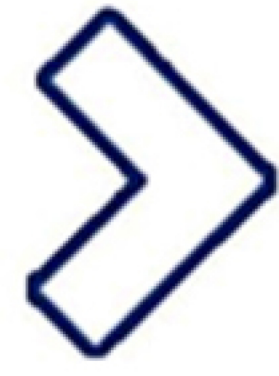


3. Place on the probe and lock

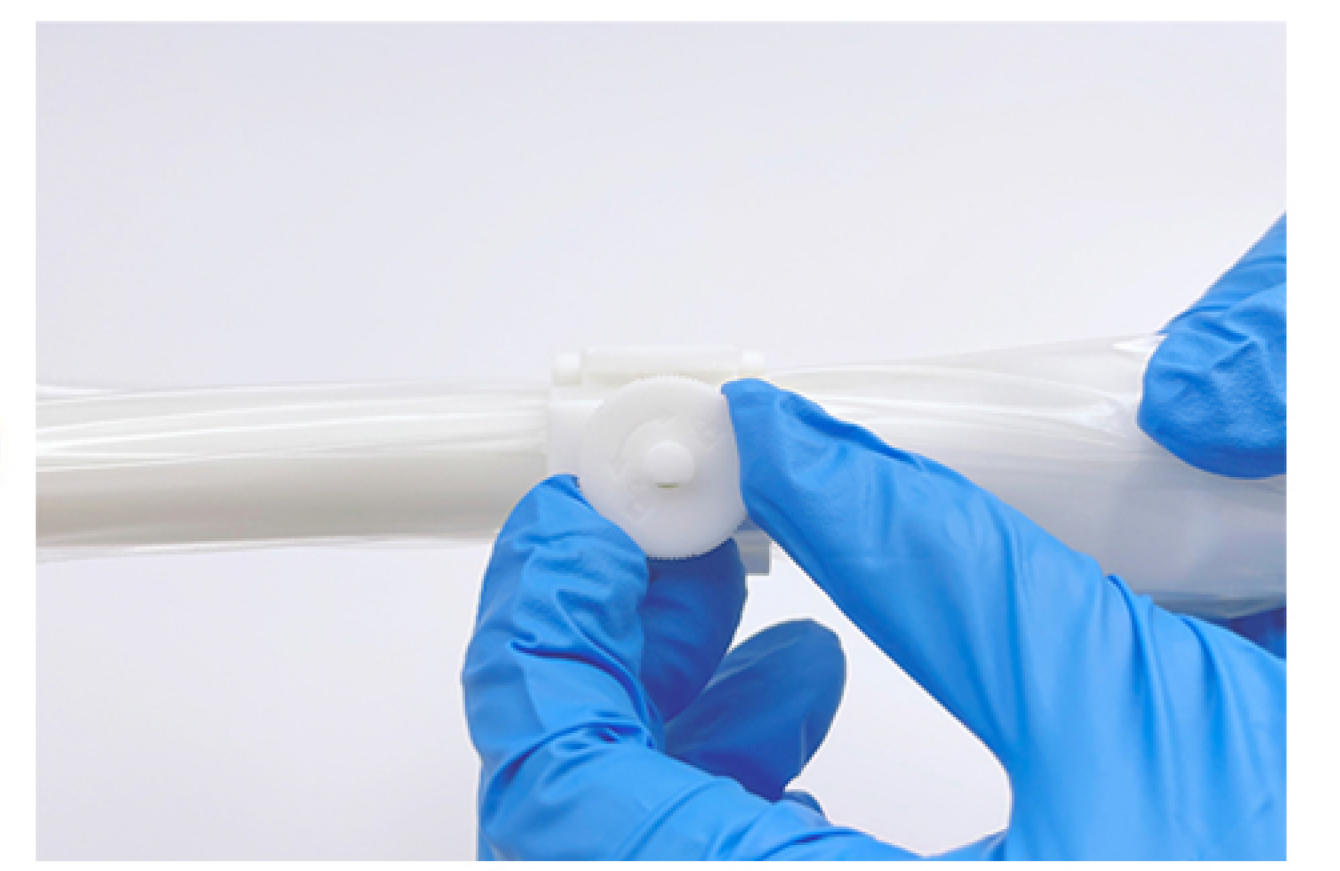


4. When in place rotate the locking screw clockwise to secure to the probe

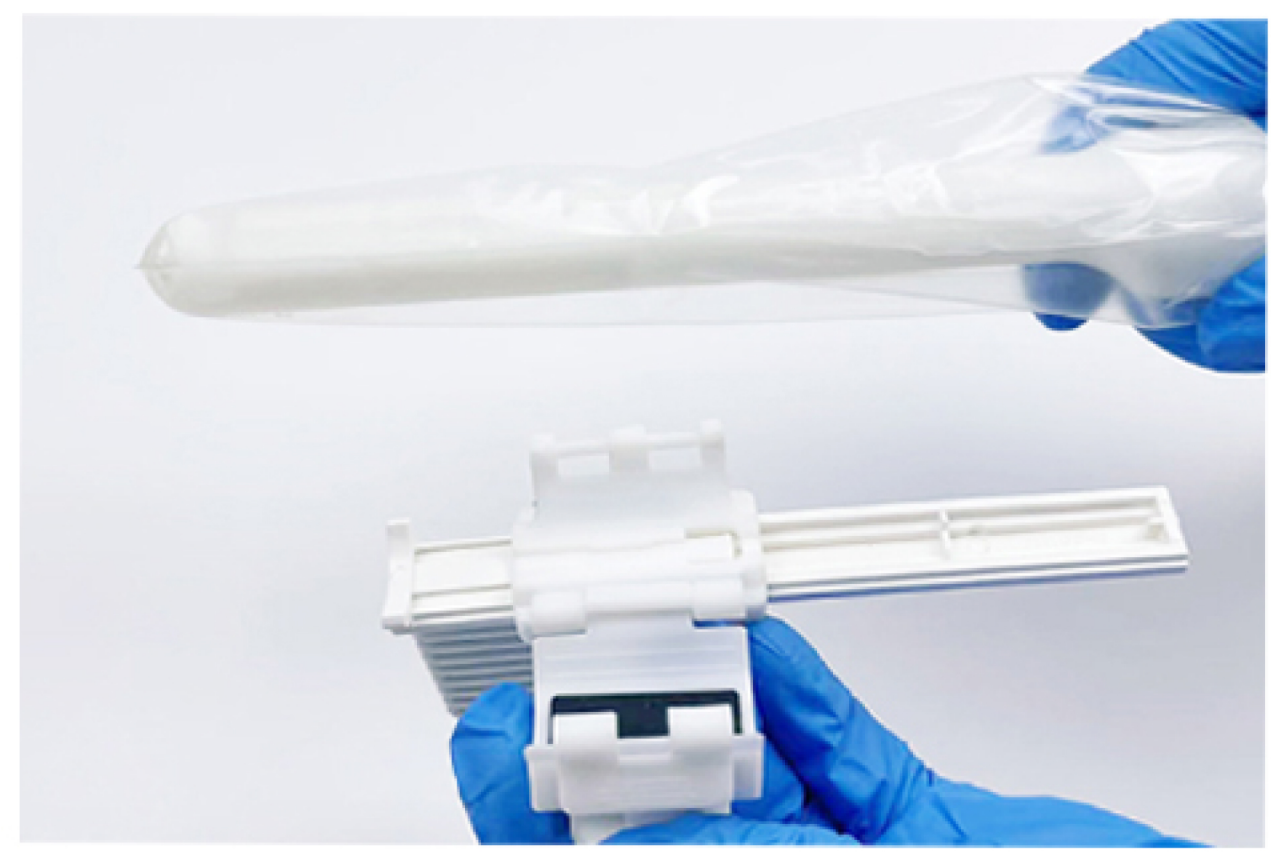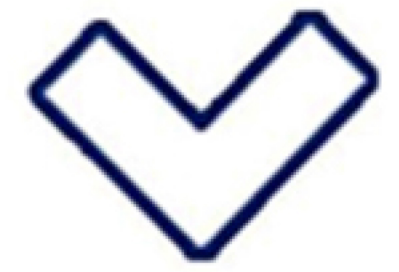

8. Press the left-side of the locking arm to release guide

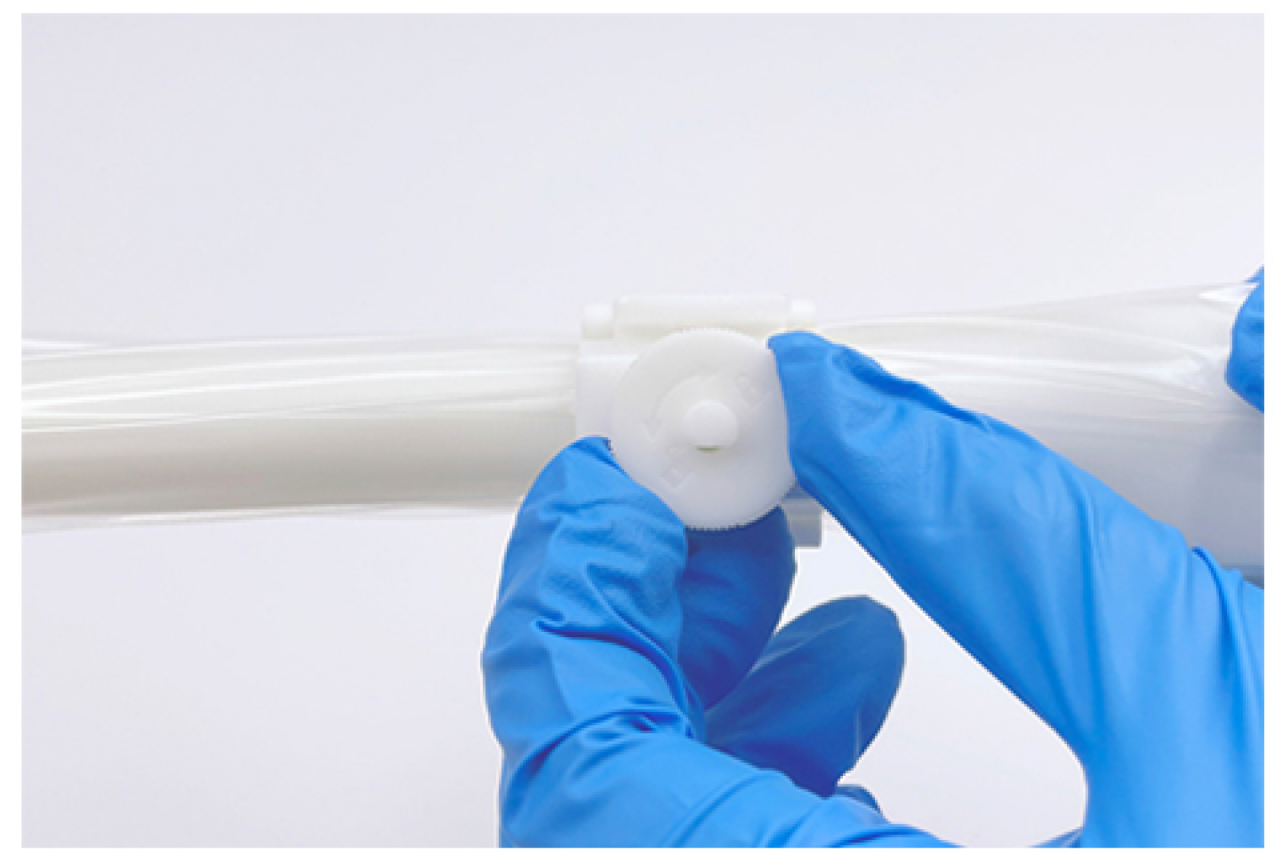


7. Rotate the locking screw counterclockwise fully when procedure is complete

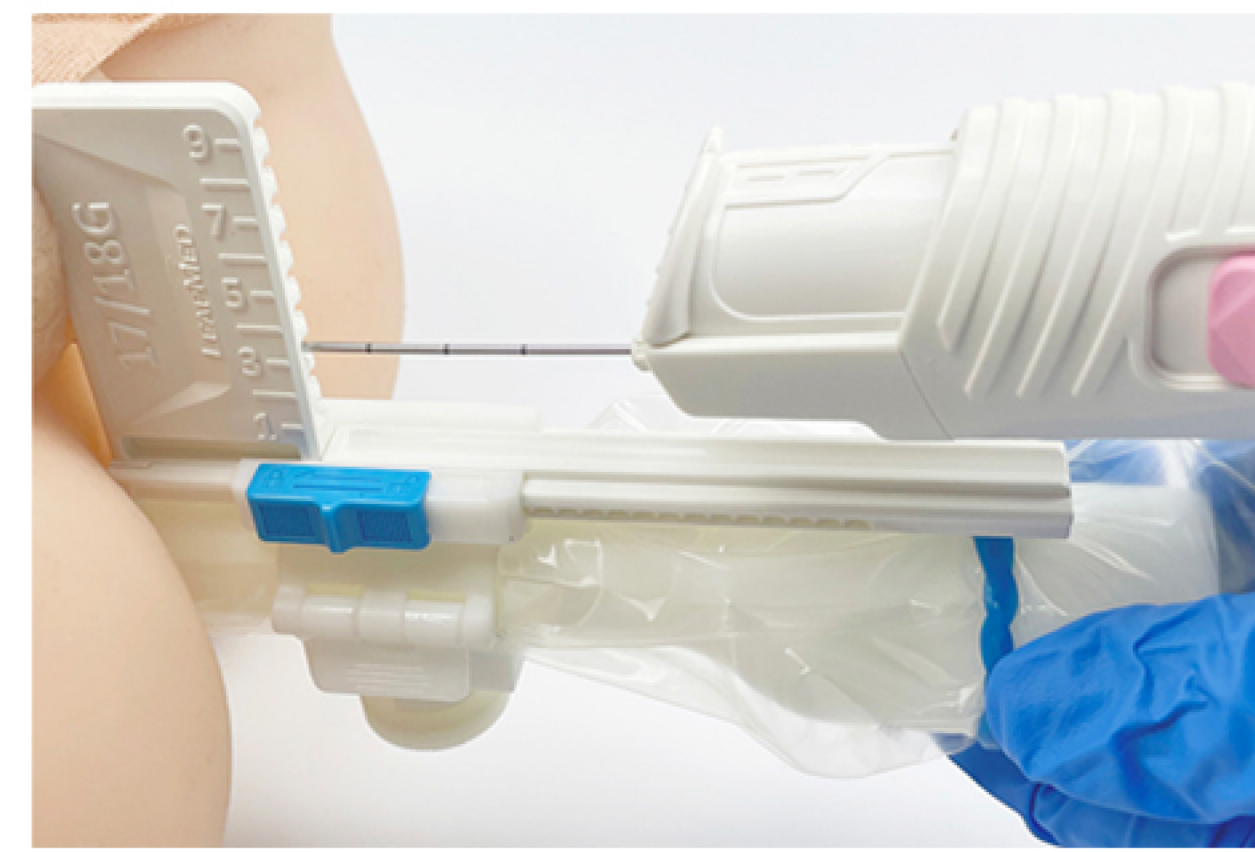

6. Insert the instrument through a channel corresponding to the target using on-screen visualization



5. Insert probe, place channel guide against rectum, and lock the channel guide

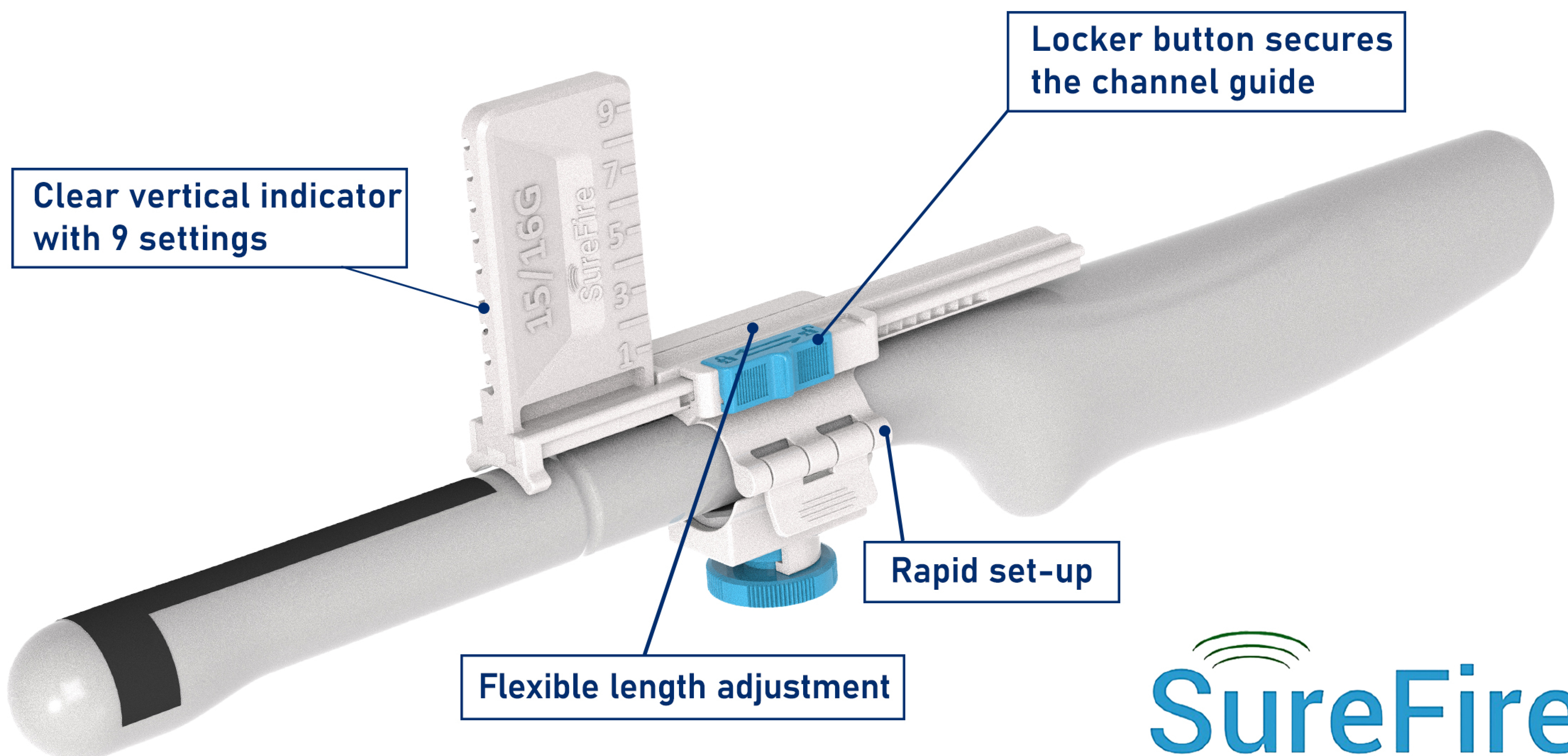

SureFire